## Porto Laparoscopico ADAPT™

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Pediatric Port - REF 43507 ▶ 3/5mm x 70mm  
Short Port - REF 40507 ▶ 5mm x 70mm

Figure 1

Figure 2

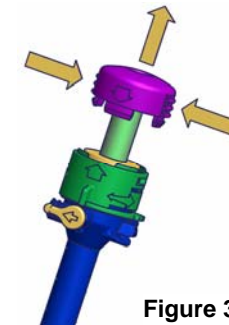
Figure 3

Figure 4

LATEX FREE

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.

TAUT<sup>INC.</sup>

2571 Kaneville Court · Geneva, IL 60134  
630/232-2507 · 800/231-TAUT(8288)

www.taut.com

## ADAPT™ Laparoscopic Port

Please review the following information thoroughly before use.

**Important:** The instructions provided are intended to facilitate the use of the ADAPT Laparoscopic Port. They are not intended to be an implication of trocar introduction methods. Properly following the instructions is essential to the safe and effective use of the product. Failure to do so may result in adverse surgical results.

**Product Indications:** The ADAPT Laparoscopic Port is indicated for use in thoracic, abdominal, and gynecologic minimally invasive surgical procedures to provide a pathway for the introduction of endoscopic surgical devices.

**Contraindications:** Where minimally invasive techniques are contraindicated, other methods and instrumentation should be employed.

**Instructions for Use:** Before using the ADAPT Laparoscopic Port, confirm the compatibility of all instruments and accessories. Prepare the patient in accordance with proper surgical techniques before insertion of the ADAPT Laparoscopic Port.

1. Remove the device from the package using standard sterile technique. To prevent damage, care should be taken not to allow the device to fall into the sterile field.
2. Insert the obturator into the cannula until both components lock securely together. Confirm that the tip is properly aligned.
3. Perform initial insufflation using standard surgical techniques.
4. Make an adequate incision at the puncture site to ease insertion. Introduce the obturator and cannula through the incision and apply continuous but controlled pressure with a deliberate rotating motion of approximately 180°. Once the primary port is in place, insert all secondary ports under direct visualization.
5. Once proper placement of the device has been established in the abdominal or thoracic cavity, press the locking tabs to remove the obturator, maintaining the position of the cannula. When obturator is removed, the internal valve automatically closes.
6. Insufflation may be maintained by attaching insufflation tubing to the stopcock on the cannula. When an instrument is inserted in the cannula, an outer seal maintains insufflation. The seal mechanism also maintains insufflation in the absence of an instrument in the cannula.
7. If additional stabilization of the cannula is required, the enclosed anchoring button can be utilized. Insert the cannula through the button before placement in the patient, and advance the button up the shaft of the cannula until it reaches the head. Once the port is placed at the desired depth in the patient, slide the anchoring button back down the cannula until it contacts the patient. Standard suturing material may then be threaded through the anchoring button.
8. For rapid desufflation or removal of a tissue sample during the procedure, use your index finger to compress the locking lever located on the head of cannula. While maintaining locking lever compression, turn the blue end of the cannula head counter-clockwise 90°. This will completely open the internal valve assembly. Once tissue sampling or desufflation is complete, rotate the blue end of cannula head clockwise, until it returns to its original locked position.

### Warnings and Precautions:

- The complications and risks of minimally invasive procedures should be reviewed prior to their performance through consultation of medical literature. Only persons having adequate training and familiarity should perform minimally invasive procedures.
- Although precautionary measures involved in obturator insertions must be observed, the uniquely designed obturator tip is intended to reduce the possibility of injury to surrounding anatomic structures.
- Due to the variance in diameter of minimally invasive instruments, compatibility should be confirmed before initiating any minimally invasive procedure.
- Desufflation of the abdominal or thoracic cavity may result from introducing instruments with a smaller diameter than the ADAPT Laparoscopic Port.
- Discontinue advancement of the obturator device once complete entry has been established into the abdominal or thoracic cavity in order to avoid injury to surrounding structures.
- Insertion and removal of instruments through the cannula should be done carefully to prevent unintentional damage to the internal seals, which may compromise insufflation. To ensure smooth insertion, avoid inserting instruments at an angle. Always attempt to insert instrumentation straight down the cannula.
- After removing the ADAPT Laparoscopic Port from the cavity, always inspect the insertion site for hemostasis. If hemostasis is not present, appropriate techniques should be used to achieve hemostasis.
- Properly discard using accepted disposal practices.
- **DO NOT RESTERILIZE.** Further processing may diminish the effectiveness of the device, which may result in unintended injury.

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