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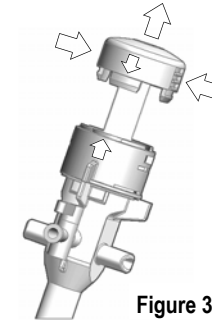
Figure 3

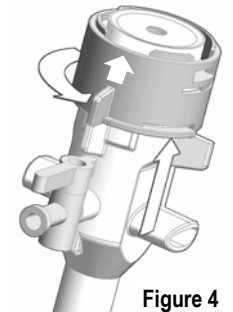
Figure 4

CAUTION: Federal (USA) law restricts this device to sale or on the order of a physician.

TAUT INC.®

800-231-8288 / www.taut.com
2571 Kaneville Court • Geneva, IL 60134
630-232-2507 / Fax: 630-232-8005

ADAPT™ Open Access Port

Please review the following information thoroughly before use.

Important: The instructions provided are intended to facilitate the use of the ADAPT Open Access Port. They are not intended to be an implication of trocar introduction methods. Properly following the instructions is essential to the safe and effective use of the product. Failure to do so may result in adverse surgical results.

Product Indications: The ADAPT Open Access Port is indicated for use in thoracic, abdominal, and gynecologic minimally invasive surgical procedures to provide a pathway for the introduction of endoscopic surgical devices.

Contraindications: Where minimally invasive techniques are contraindicated, other methods and instrumentation should be employed.

Instructions for Use: Before using the ADAPT Open Access Port, confirm the compatibility of all instruments and accessories. Prepare the patient in accordance with proper surgical techniques before insertion of the ADAPT Open Access Port.

1. Remove the device from the package using standard sterile technique. To prevent damage, care should be taken not to allow the device to fall into the sterile field.
2. Align the two white arrows as you insert the obturator into the cannula until both components lock securely together (see Figure 1). Confirm that the tip is properly aligned.
3. Make an adequate incision at the desired site. Carefully dissect the tissue down to, and through, the desired tissue plane.
4. Pass anchoring sutures of an adequate size through each fascial edge. Hold the sutures upward and apart.
5. Unlock the *Softseal* Adjustable Plug by rotating the locking tabs counter-clockwise, and slide the plug to the top of the cannula. Insert the port into the incision, and slide the *Softseal* Plug down into the incision. Pull and wrap the sutures firmly around the suturing posts on the plug, forcing the plug firmly into the incision to minimize gas leakage.
6. Position the cannula at the desired depth and lock the *Softseal* plug into place by rotating the locking tabs on the plug clockwise (see figure 2).
7. Once proper placement of the device has been established, press the locking tabs on the obturator to remove it. When obturator is removed, the internal valve automatically closes (see figure 3)
8. To insufflate, attach insufflation tubing to the stopcock on the cannula and open the stopcock.
9. For rapid desufflation or removal of a tissue sample during the procedure, use your index finger to compress the locking lever located on the head of cannula. While maintaining locking lever compression, turn the blue end of the cannula head counter-clockwise 90° (see Figure 4). This will completely open the internal valve assembly. Once tissue sampling or desufflation is complete, rotate the blue end of cannula head clockwise, until it returns to its original locked position.

Warnings and Precautions:

- The complications and risks of minimally invasive procedures should be reviewed prior to their performance through consultation of medical literature. Only persons having adequate training and familiarity should perform minimally invasive procedures.
- Although precautionary measures involved in obturator insertions must be observed, the uniquely designed obturator tip is intended to reduce the possibility of injury to surrounding anatomic structures.
- Due to the variance in diameter of minimally invasive instruments, compatibility should be confirmed before initiating any minimally invasive procedure.
- Desufflation of the abdominal or thoracic cavity may result from introducing instruments with a smaller diameter than the ADAPT Open Access Port. An ADAPT Laparoscopic Reducer Cap must be used if smaller diameter instruments are to be inserted through the cannula.
- Discontinue advancement of the obturator device once complete entry has been established into the abdominal or thoracic cavity in order to avoid injury to surrounding structures.
- Insertion and removal of instruments through the cannula should be done carefully to prevent unintentional damage to the internal seals, which may compromise insufflation.
- After removing the ADAPT Open Access Port from the cavity, always inspect the insertion site for hemostasis. If hemostasis is not present, appropriate techniques should be used to achieve hemostasis.
- Properly discard using accepted disposal practices.
- **DO NOT RESTERILIZE.** Further processing may diminish the effectiveness of the device, which may result in unintended injury.

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