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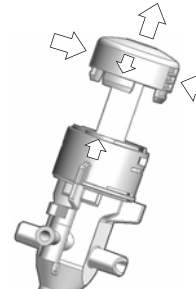
Figure 3

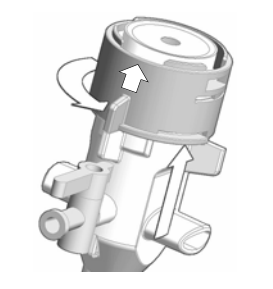
Figure 4

CAUTION: Federal (USA) law restricts this device to sale or on the order of a physician.

TAUT INC.®

800-231-8288 / www.taut.com
2571 Kaneville Court • Geneva, IL 60134
630-232-2507 / Fax: 630-232-8005

ADAPT™ Balloon Open Access Port

Please review the following information thoroughly before use.

Important: The instructions provided are intended to facilitate the use of the ADAPT Balloon Open Access Port. They are not intended to be an implication of trocar introduction methods. Properly following the instructions is essential to the safe and effective use of the product. Failure to do so may result in adverse surgical results.

Product Indications: The ADAPT Balloon Open Access Port is indicated for use in thoracic, abdominal, and gynecologic minimally invasive surgical procedures to provide a pathway for the introduction of endoscopic surgical devices.

Contraindications: Where minimally invasive techniques are contraindicated, other methods and instrumentation should be employed.

Instructions for Use: Before using the ADAPT Balloon Open Access Port, confirm the compatibility of all instruments and accessories. Prepare the patient in accordance with proper surgical techniques before insertion of the ADAPT Balloon Open Access Port.

1. Remove the device from the package using standard sterile technique. To prevent damage, care should be taken not to allow the device to fall into the sterile field.
2. Insert the obturator into the cannula until both components lock securely together (See Figure 1).
3. Make an adequate incision at the desired site. Carefully dissect the tissue down to, and through, the desired tissue plane.
4. Unlock the *Foamseal* Adjustable Plug by twisting the locking tabs clockwise, and slide the plug to the top of the cannula. Insert the ADAPT Balloon Open Access Port into the incision until the balloon portion at the distal tip is placed within the desired space.
5. Fill the provided syringe with **10CC OF STERILE LIQUID OR 15CC OF AIR**. Attach the syringe to the balloon inflation port and inflate slowly. **CAUTION: OVER-INFLATION OF THE BALLOON MAY CAUSE IT TO RUPTURE. NOTE THAT 10CC OF FLUID OR 15CC OF AIR WILL COMPLETELY INFLATE THE BALLOON AND FURTHER INFLATION IS UNNECESSARY AND WILL INCREASE THE RISK OF BALLOON RUPTURE.**
6. Pull up on the port while pushing the *Foamseal*/plug firmly down to the skin, compressing the foam. Lock the plug into place by twisting the locking tabs clockwise (see figure 2). To unlock or reposition the *Foamseal*, simply twist the locking tabs counterclockwise.
7. Once proper placement of the device has been established, press the locking tabs on the obturator to remove it. (See Figure 3) When obturator is removed, the internal valve automatically closes.
8. To insufflate, attach insufflation tubing to the stopcock on the cannula and open the stopcock. When an instrument is inserted into the cannula, an outer seal maintains insufflation. The seal mechanism also maintains insufflation in the absence of an instrument in the cannula.
9. To remove the ADAPT Balloon Open Access Port, reattach the syringe to the balloon inflation port. Draw back on the plunger until all fluid or air has been removed and the balloon is fully deflated. Remove the Open Access Port only after the balloon has been fully deflated.
10. For rapid desufflation or removal of a tissue sample during the procedure, use your index finger to compress the locking lever located on the head of cannula. While maintaining locking lever compression, turn the blue end of the cannula head counter-clockwise 90° (See Figure 4). This will completely open the internal valve assembly. Once tissue sampling or desufflation is complete, rotate the blue end of cannula head clockwise, until it returns to its original locked position.

Warnings and Precautions:

- The complications and risks of minimally invasive procedures should be reviewed prior to their performance through consultation of medical literature. Only persons having adequate training and familiarity should perform minimally invasive procedures.
- Although precautionary measures involved in obturator insertions must be observed, the uniquely designed obturator tip is intended to reduce the possibility of injury to surrounding anatomic structures.
- Due to the variance in diameter of minimally invasive instruments, compatibility should be confirmed before initiating any minimally invasive procedure.
- Desufflation of the abdominal or thoracic cavity may result from introducing instruments with a smaller diameter than the ADAPT Balloon Open Access Port. An ADAPT Laparoscopic Reducer Cap must be used if smaller diameter instruments are to be inserted through the cannula.
- Discontinue advancement of the obturator device once complete entry has been established into the abdominal or thoracic cavity in order to avoid injury to surrounding structures.
- Insertion and removal of instruments through the cannula should be done carefully to prevent unintentional damage to the internal seals, which may compromise insufflation.
- After removing the ADAPT Balloon Open Access Port from the cavity, always inspect the insertion site for homeostasis. If homeostasis is not present, appropriate techniques should be used to achieve homeostasis.
- Properly discard using accepted disposal practices.
- **DO NOT RESTERILIZE.** Further processing may diminish the effectiveness of the device, which may result in unintended injury.

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